


Steps to Pair the 3M™ Oral Care Portal and 3Shape Communicate™

Quick Start Guide


1. Log in to 3Shape Communicate at portal.3shapecommunicate.com.
2. Navigate to “Connect to a practice or Lab” and enter customortho@mmm.com as the email.
3. Send the connection request.
4. Log in to Oral Care portal at OralCare.3M.com.
5. Access Manage Account by clicking the arrow at the top right of the screen.
6. Click on your account name and then on the settings tab.

Steps to Pair the 3M™ Oral Care Portal and 3Shape Communicate™ (cont.)

7. On the Settings tab select Scan, check the 3Shape TRIOS® box and then select Add Account.

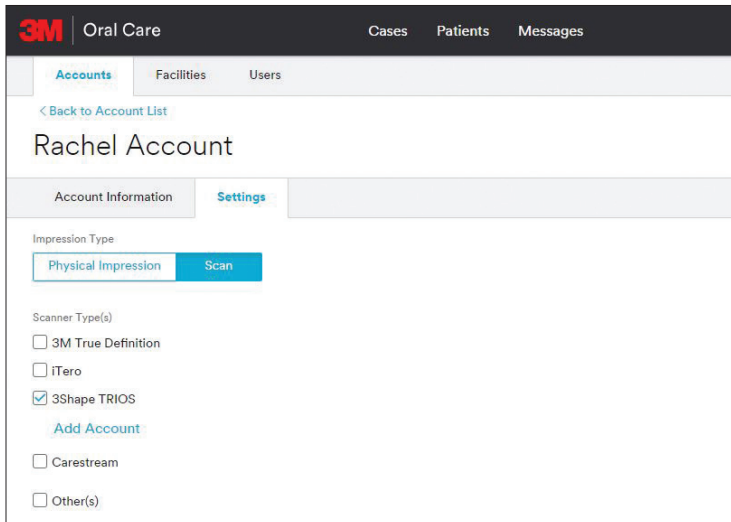

8. Log in to your 3Shape Communicate account and authorize the connection.

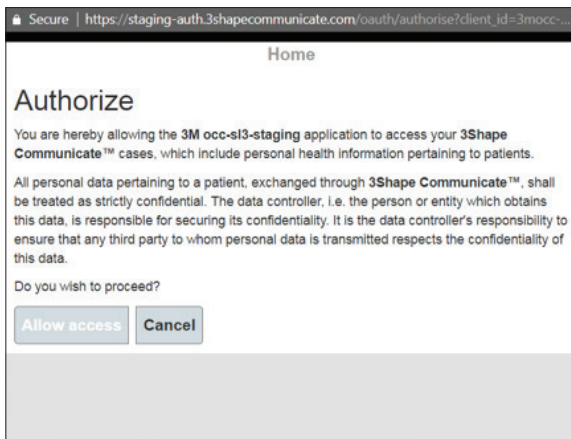

9. Select Save as Defaults.
10. On the TRIOS scanner navigate to Local Lab and click on Refresh. The 3M Oral Care Lab should appear in the list.
11. This completes the pairing.



3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA
3M.com/ortho

For more information contact
3M Oral Care at 1-800-276-8789

3M and Clarity are trademarks of 3M. All other trademarks are the property of their respective owners. © 3M 2018. All rights reserved. Please recycle. (Rev. 10/2018)